

Tel. / Fax: 0375 480032 Cell. Luigi: 320 4791252 Cell. Luca: 348 4836037 Via Pastrengo, 13 – fraz. Cappella 26041 – Casalmaggiore (CR) Italy Cod. Fisc. / P. IVA: 01462360197 Cremona C.C.I.A.A. 173598

E-mail: info@odlsnc.it | acquisti@odlsnc.it | commerciale@odlsnc.it - Website: www.odlsnc.it

Elettromedicali | Carpenteria Leggera | Lavorazione Materie Plastiche

# INSTRUCTIONS for INSTALLATION and MAINTENANCE "LED VIEWING BOXES"

## **INSTALLATION:**

Hang the viewing box to the wall using the special key-shaped holes on the back (Pict. 5).


## USE:

#### · Power on:

- Connect the power socket with the supplied cable (Pict. 1).
- Set the main switch to 1 (Pict. 1); the dot will light up on the display (Pict. 2).
- Manual: the sector is switched on by pressing the "on / off" button (Pict. 2); the display will show the degree of regulation memorized in the last switching on from 0 to 9 (Pict. 3).
- If the "Automatic" mode is desired, press the button (Pict. 2); the "A" for Automatic will appear on the display (Pict. 4); the sector will light up by contacts, when the plate is inserted (Pict. 6).

## · Regulation:

Increase or decrease the light intensity using the buttons (Pict. 2).

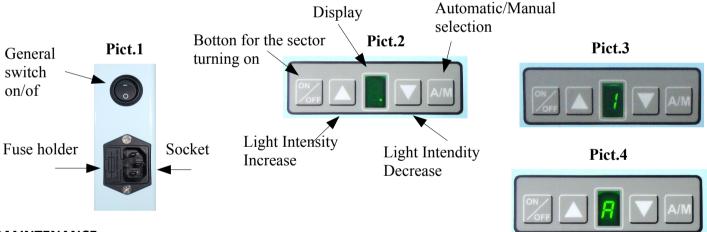

# **MAINTENANCE:**

- To clean it use soft cloth and products that do not contain alcohol or solvents.
- To reach the inner parts, unscrew the rear panel (Pict. 5), operation to be carried out by specialized personnel (screws on the edge).

