

Tel. / Fax: 0375 480032 Cell. Luigi: 320 4791252 Cell. Luca: 348 4836037

Via Pastrengo, 13 – fraz. Cappella 26041 – Casalmaggiore (CR) Italy Cod. Fisc. / P. IVA: 01462360197 Cremona C.C.I.A.A. 173598

E-mail: info@odlsnc.it | acquisti@odlsnc.it | commerciale@odlsnc.it - Website: www.odlsnc.it

Elettromedicali | Carpenteria Leggera | Lavorazione Materie Plastiche

## INSTRUCTIONS for INSTALLATION and MAINENANCE "OPTOTYPE"

## **INSTALLATION:**

- Attach 2 dowels to the wall and apply 2 hook bolts
- Hang the optotype to the wall using the special key-shaped holes on the back

If you want to fix it to the wall for enhanced security:

- · Attach 2 dowels to the wall
- Remove the plexiglas plate
- Fix the optotype to the wall with 2 dowels screws through the special key-shaped holes on the back

## **MAINTENANCE:**

The optotype does not need any particular maintenance.

• To clean it use soft cloth and products that do not contain alcohol or solvents

To reach the inner parts, perform the following operations:

- · Take off tension to the optotype
- Unscrew and remove the upper frame, even pulling the corners
- Remove the Plexiglas plate
- To reassemble slide the plexiglas plate, screw on the frame and apply the corners

